


ORIO-C

PEEK-Optima®

Anterior Cervical Cage System

ORIO-C Cervical Cage System Surgical Technique

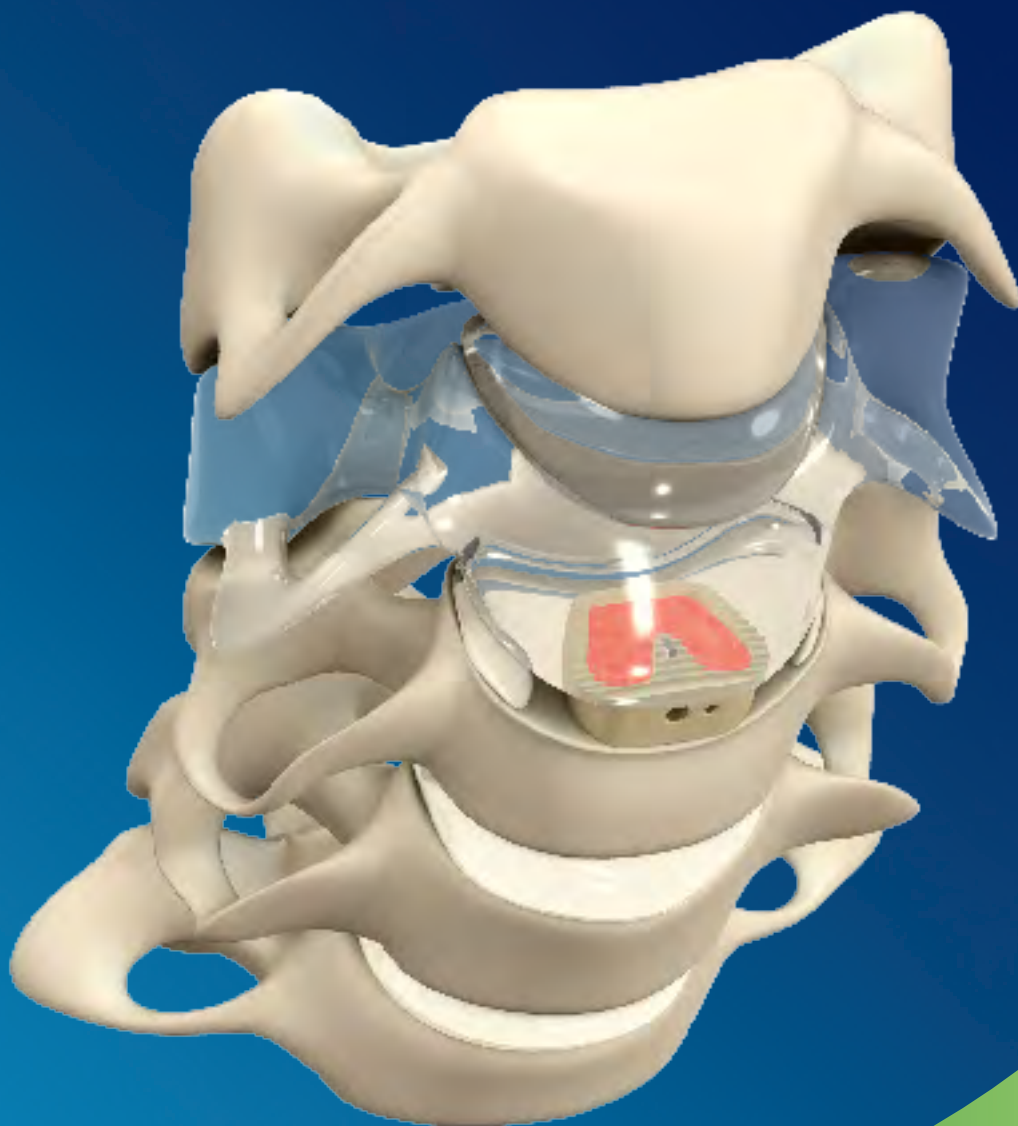

CERVICAL CAGE SYSTEM SURGICAL TECHNIQUE

Techniques described by:

- **Anis Mekhail, MD**
Assistant Clinical Professor at University of Illinois at Chicago
Palos Heights, Illinois
- and
- **Steven Mather, MD**
North American Spine Society
Downers Grove, Illinois

Disclaimer

This document is intended exclusively for experts in the field, i.e. physicians in particular, and is expressly not for the information of laypersons.

The information on the products and/or procedures contained in this document is of a general nature and does not represent medical advice or recommendations. Since this information does not constitute any diagnostic or therapeutic statement with regard to any individual medical case, individual examination and advising of the respective patient are absolutely necessary and are not replaced by this document in whole or in part.

In the event that this document could be construed as an offer at any time, such offer shall not be binding in any event and shall require subsequent confirmation in writing.

Table of Contents


01. Implant Design Rationale	4
02. Implant Ordering Information	5
03. Instrument Ordering Information	6
04. Surgical Technique	7
05. Instruction for Use	13

01. Implant Design Rationale


ORIO-C is a system of curved implants and instruments designed for anterior cervical interbody fusion (ACIF). ORIO-C was developed for the following purposes:

- To distract the disc space and restore normal disc height and physiological lordosis, thereby also widening the foramina
- To preserve the integrity of the vertebral body endplates
- To provide an optimal implant/endplate interface, thus considerably limiting the risk of subsidence into the adjacent vertebrae
- To stabilize the pathologically unstable segment
- To support bone growth through the implant


The proper positioning of the ORIO-C cage is checked with a radiopaque tantalum marker and a titanium keel.

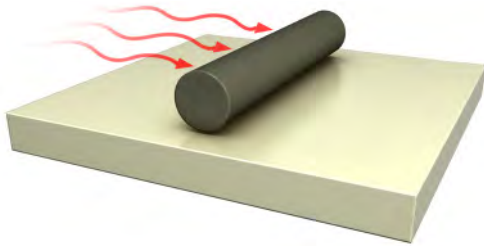


Thanks to the PEEK-Optima® radiotransparency, the ORIO-C cage is invisible on X-Rays for a clear assessment of bone fusion.

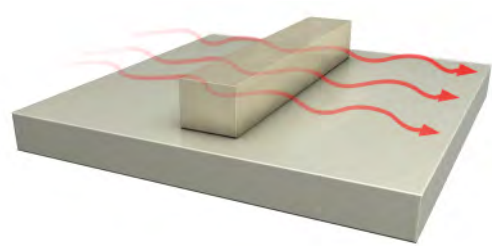

Placement of radiopaque marker in radiotranslucent PEEK-Optima®

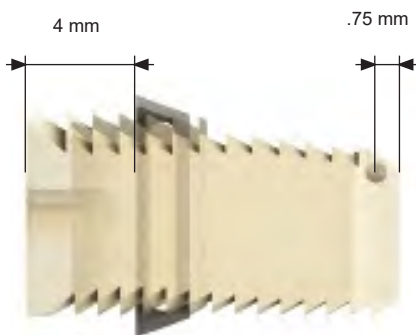

02. Implant Ordering Information

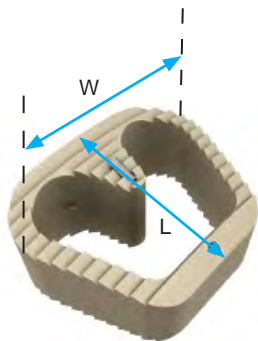

Anatomical curve of cranial and caudal surfaces for better fit to endplates and optimized load-sharing



ORIO Cervical PEEK-Optima® Cage, Anatomical w/Anchor	Length	Width	Height	Catalog n°
Size Small	12 mm	15 mm	4 mm	C7100-24
Size Small	12 mm	15 mm	5 mm	C7100-25
Size Small	12 mm	15 mm	6 mm	C7100-26
Size Small	12 mm	15 mm	7 mm	C7100-27
Size Small	12 mm	15 mm	8 mm	C7100-28
Size Large	14 mm	17 mm	4 mm	C7100-44
Size Large	14 mm	17 mm	5 mm	C7100-45
Size Large	14 mm	17 mm	6 mm	C7100-46
Size Large	14 mm	17 mm	7 mm	C7100-47
Size Large	14 mm	17 mm	8 mm	C7100-48

03. Instrument Ordering Information



Large Graft Compactor

7000-1-0046



Cage Jig

7000-1-0076



Small Graft Compactor

7000-1-0045



Final Impactor

7000-1-0525



Anchor Pusher

7000-1-0530



Depth Gauge

7000-1-0540



Cage Inserter/Impactor

7000-1-0506



Trial Cage Inserter

7000-1-0760



Cage Forceps

7000-1-0552



Trial 12 x 15 x 4 mm
Trial 12 x 15 x 5 mm
Trial 12 x 15 x 6 mm
Trial 12 x 15 x 7 mm
Trial 12 x 15 x 8 mm

7000-1-0145
7000-1-0153
7000-1-0161
7000-1-0169
7000-1-0177



Trial 14 x 17 x 4 mm
Trial 14 x 17 x 5 mm
Trial 14 x 17 x 6 mm
Trial 14 x 17 x 7 mm
Trial 14 x 17 x 8 mm

7000-1-0345
7000-1-0353
7000-1-0361
7000-1-0369
7000-1-0377



Curette

7000-1-0800



Anchor Extractor

7000-1-0900

Implant / Instrument Case

7000-2-0000

Implant Caddy Universal

7000-2-0003


04. Surgical Technique

Preoperative Planning

1

Prior to surgery, the desired surgical approach must be determined and the appropriate ORIO cage height estimated.

Cage size can be determined by measuring the distracted disc space during the operation using the trial cages.

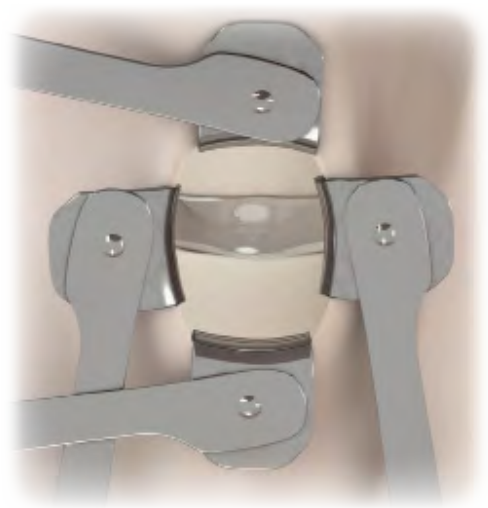

Disc Exposure

2

Expose the affected disc and adjacent vertebral bodies through an anterolateral incision in the cervical spine.



Disc Space Preparation


3

A wide anterior approach will demonstrate the lateral limits of the uncovertebral joints and guide the anterior decompression after the posterior annulus is incised. Use rongeurs to remove the disc material through this window.

Distraction

4

Distract the segment. Distraction of the segment is essential for restoration of disc height and for providing good access to the intervertebral space for subsequent optimal preparation of the endplates. A Caspar Distractor is recommended for distraction.

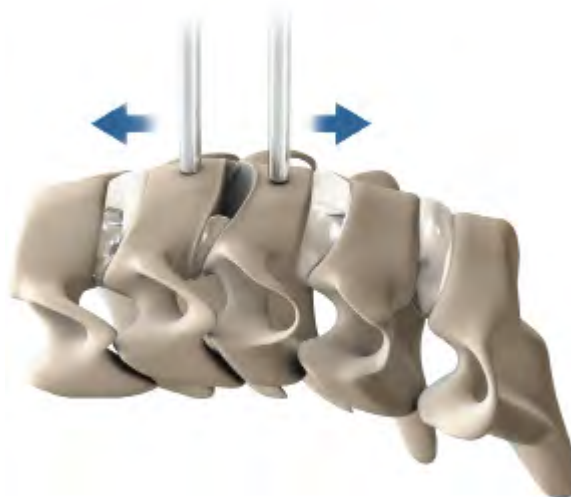

Vertebral Body Endplate Preparation


5

The selected preparation technique will depend on the surgeon's own preference and experience. Either of the two following techniques may be used:

1) The first technique preserves the cortical bone beneath the cartilaginous layer and the natural shape of the bone, thereby increasing resistance to subsidence into the adjacent vertebrae. Remove the cartilaginous layers from the surface of the vertebral endplates using the Curette until bleeding bone is attained. Clean the endplates. Adequate cleaning of the endplates is important for the vascular supply to the bone graft. Excessive cleaning, however, may weaken the endplates due to removal of the denser bone of the endplate.

2) The second technique involves removal of the cartilaginous layer and part of the cortical bone to produce flat surfaces and thus provide the optimum surface contact area between the bone structure and the implant.



 *Note : In both cases, the removal of any osteophytes is very important for achieving complete decompression of the neural structures and avoiding the risk of partial compression after implant insertion.*

Use of the Trial Cage to Determine the Implant Size

6

Selection of the Trial Cage will be based on the height of the intervertebral space determined preoperatively, the preparation technique and the patient's anatomy.

The Depth Gauge can be used to determine the disc depth and whether to use the small (SM) or large (LG) footprint ORIO-C Implant/Trial Cage.



ORIO-C Trial Cage/Holder Assembly

7

Screw the Trial Cage Inserter into the Trial Cage. For the sake of clarification, the Trial Cages are marked with "CRANIAL" and "CAUDAL" etchings. If necessary, rotate the Trial Cage 180°.

Repeat with different Trial Cage sizes if necessary, until proper fitting is achieved.



Cage Preparation (optional)

8

The surgeon may or may not decide to use the cage stabilizing anchor/fin. To remove the anchor, place the cage on the Anchor Pusher platter, then squeeze the arms to slide the anchor through the cage. Insert the Anchor Extractor through the slot in the Anchor Pusher platter to grasp and pull the anchor out of the cage.


While using the Anchor Pusher, the surgeon should always avoid excessive tightening once the hammer is in contact with the implant to avoid any damage to the cage teeth.



Cage Attachment

9

Select the ORIO-C cage size determined in step 7. Slide the sleeve of the Cage Inserter up to reveal the threaded shaft tip. Start to thread it into the cage. Once a couple of threads are engaged, slide the Cage Inserter sleeve down to insert the side pin into the side hole of the cage, then finish tightening the thread.

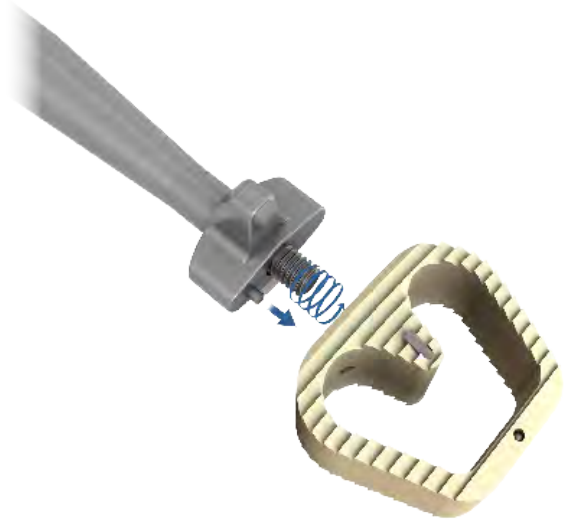

Filling of the Cage with Autologous Bone

10

Insert the cage, attached to the Cage Inserter into the correct side of the Cage Jig, depending on the cage size (SM or LG).

Fill the packing block with autologous bone and using the Small (SM) or Large (LG) Compactor, pack the cage completely with autologous bone.



ORIO-C Cage Insertion

11


Check the CRANIAL/CAUDAL marking of the Cage Inserter and insert the cage into the disc space. The Cage Inserter shoulder will prevent over-insertion of the cage and will position it at a depth of 1 mm from the anterior surfaces of the vertebrae.

Remove the Cage Inserter by holding the sleeve and turning the handle counterclockwise to unthread the cage.

The optimal position for the ORIO-C Cage is centered within the periphery of the vertebral endplates.

Further positioning of the ORIO-C Cage can be achieved if necessary with the Final Impactor. The Final Impactor shoulder will prevent over-insertion of the cage into the disc space.

 *Note : Verify the optimal position of the cage under the image intensifier.*

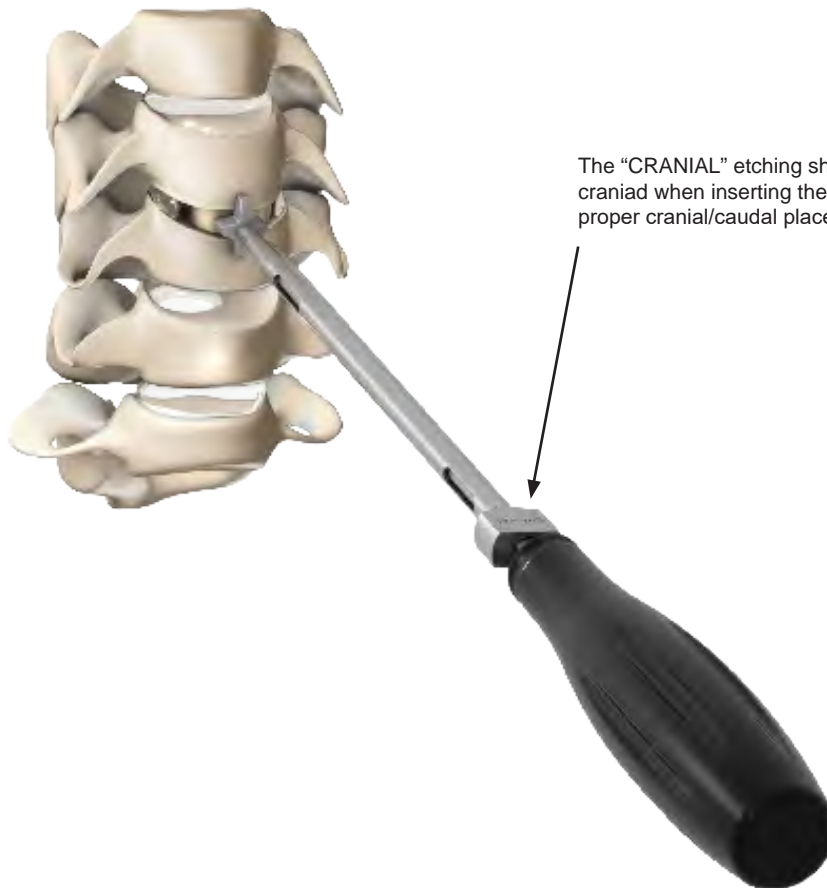

The "CRANIAL" etching should be oriented craniad when inserting the cage to ensure proper cranial/caudal placement.

Supplemental Fixation

ORIO-C Cages must be used with supplemental fixation to enhance the biomechanical stability of the motion segment. The addition of anterior plate fixation offers many benefits such as: resistance to graft displacement, a reduced incidence of pseudarthrosis related to micro motion at the graft vertebral body interface, maintaining anterior cervical alignment when multi-level discectomies or corpectomies are performed, and a decreased reliance on prolonged external bracing.

Anterior cervical plating is required to stabilize the spine for tumor, trauma, deformity, degenerative disc disease and other forms of cervical instability. Supplemental fixation also improves the stability of the ORIO-C cages.

See SpineCraft's [ALTUM Anterior Cervical Plate Surgical Technique](#). This instrument set includes a Cage Inserter Drill Guide instrument, which allows for integration between the two systems.

Cage Removal

PEEK-Optima® composite material has mechanical properties similar to cortical bone. Excessive leverage against the handle of the insertion tool can split or break the cage. If a cage breaks during insertion, it should be replaced.

Distraction of the involved level should be applied. The broken cage then can be grasped and removed using a Kocher clamp or hemostat. A cage can also be removed in fragments, if necessary.

Please note that the cages should be retrieved as carefully as possible in order to keep the implant and surrounding tissue intact. Also, please document descriptive information about the gross appearance of the device in situ, as well as descriptions of the removal methods, e.g., intact or in pieces.

05. Instructions for Use

ORIO Intervertebral Body Fusion Cage System

INSTRUCTIONS FOR USE

CAUTION: USA law restricts this device to sale by or on the order of physician.

1. PRODUCT HANDLING

The implants and instruments used as a part of these systems are provided nonsterile and should be stored in their original packaging or placed within the aluminium sterilization container provided. Implants and instruments should be stored in such a manner when not in use, and until they are cleaned and sterilized according to the recommended guidelines listed below. Protect implants from contact with objects that may damage the surface finish. Inspect each implant prior to use for visual damage.

2. PRODUCT DESCRIPTION AND IMPLANT MATERIALS

The ORIO intervertebral body fusion cage is a single component spinal device made from PEEK-Optima® polymer (polyetheretherketone). The implant is available in a range of sizes and lordotic angles to fit the patient's anatomical and physiological requirements. The implants have cavities to accept packing of bone graft. The entire structure is radiolucent so that healing can be assessed by normal radiographic methods. Additionally, radiotherapy can be performed immediately after surgery.

3. INDICATIONS

ORIO intervertebral body fusion cervical cages are indicated for use in skeletally mature patients with degenerative disc disease (DDD) of the cervical spine with accompanying radicular symptoms at one disc level. DDD is defined as discogenic pain with degeneration of the disc confirmed by patient history and radiographic studies. ORIO intervertebral body fusion cervical cages are used to facilitate intervertebral body fusion in the cervical spine and are placed via an anterior approach at the C3 to C7 disc levels using autograft bone. ORIO intervertebral body fusion cages are to be used with supplemental fixation. Patients should have at least six (6) weeks of non-operative treatment prior to treatment with an intervertebral cage.

The ORIO intervertebral body fusion lumbar cage is indicated for intervertebral body fusion procedures in skeletally mature patients with degenerative disc disease (DDD) of the lumbar spine at one or two contiguous levels from L2-S1. Degenerative disc disease is defined as discogenic back pain with degeneration of the disc confirmed by history and radiographic studies. These DDD patients may have up to Grade 1 spondylolisthesis or retrolisthesis at the involved level(s). ORIO intervertebral body fusion lumbar cage implants are to be used with autogenous bone graft and implanted via a transforaminal, open posterior or lateral approach. The ORIO intervertebral body fusion lumbar cages are to be used with supplemental fixation. Patients should have at least (6) months of non-operative treatment prior to treatment with an intervertebral cage.

4. PATIENT SELECTION

In selecting patients for internal fixation devices, the following factors can be of extreme importance to the eventual success of the procedure:

A. The patient's occupation or activity. If the patient is involved in an occupation or activity that includes heavy lifting, muscle strain, twisting, repetitive bending, stooping, running, substantial walking, or manual labor, he/she should not return to these activities until the bone is fully healed. Even with full healing, the patient may not be able to return to these activities successfully.

B. A condition of senility, mental illness, alcoholism, or drug abuse. These conditions, among others, may cause the patient to ignore certain necessary limitations and precautions in the use of the appliance, leading to implant failure or other complications.

C. Certain degenerative diseases. In some cases, the progression of degenerative disease may be so advanced at the time of implantation that it may substantially decrease the expected useful life of the appliance. For such cases, orthopaedic devices can only be considered a delaying technique or temporary remedy.

D. Foreign body sensitivity. The surgeon is advised that no preoperative test can completely exclude the possibility of sensitivity or allergic reaction. Patients can develop sensitivity or allergy after implants have been in the body for a period of time.

E. Smoking. Patients who smoke have been observed to experience higher

rates of pseudo-arthritis following surgical procedures where bone graft is used. Additionally, smoking has been shown to cause diffuse degeneration of intervertebral discs. Progressive degeneration of adjacent segments caused by smoking can lead to late clinical failure (recurring pain) even after successful fusion and initial clinical improvement.

5. CONTRAINDICATIONS

- Active sepsis;
- Pregnancy;
- Muscular, neurological or vascular deficiencies, which compromise the affected extremity;
- Conditions that place excessive demand on the implant (i.e. Charcot's joints, muscle deficiencies, refusal to modify post-operative physical activities, skeletal immaturity);
- Active infection in the area of proposed surgery;
- Severe osteoporosis;
- Paget's disease;
- Renal osteodystrophy;
- Advanced diabetes;
- Rheumatoid arthritis;
- Immunological suppression;
- Sustained trauma with instability;
- Fracture of the vertebra;
- Conditions requiring steroids in excess of usual doses;
- Obesity
- Signs of local inflammation,
- Fever or leukocytosis,
- Mental illness,
- Any other condition which would preclude the potential benefit of spinal implant surgery, such as the presence of tumors or congenital abnormalities, fracture local to the operating site, elevation of sedimentation rate unexplained by other diseases, elevation of white blood count (WBC), or a marked left shift in the WBC differential count,
- Suspected or documented allergy or intolerance to composite materials,
- Any case not needing a fusion,
- Any case not described in the indications,
- Any patient unwilling to cooperate with postoperative instructions,
- Patients with a known hereditary or acquired bone friability or calcification problem should not be considered for this type of surgery,
- These devices must not be used for pediatric cases, nor where the patient still has general skeletal growth.
- Spondylolisthesis unable to be reduced to Grade 1,
- Any case where the implant components selected for use would be too large or too small to achieve a successful result,
- Any case that requires the mixing of metals from two different components or systems,
- Any patient having inadequate tissue coverage over the operative site or inadequate bone stock or quality,
- Any patient in which implant utilization would interfere with anatomical structures or expected physiological performance,
- Prior fusion at the level to be treated

6. PRECAUTIONS

- **SURGICAL IMPLANTS MUST NEVER BE REUSED.** Although the device may appear undamaged, previous stresses could create imperfections that may lead to mechanical failure. Reusing an implant can potentially cause cross contamination. It is advised to utilize new implant of current design.
- **CORRECT HANDLING OF THE IMPLANT IS EXTREMELY IMPORTANT.** Familiarity with, and attention to the surgical technique recommended for this device is imperative for best results. The correct selection as well as the correct seating/placement of the implant is extremely important. SpineCraft instruments and implants should only be used in conjunction with other SpineCraft instruments and implants. The surgical technique may be obtained from the company or its representative.

Care must be taken to protect surfaces from nicks and scratches that could become focal points for failure. An implant must not be tampered with, as tampering could adversely affect the performance of the implant. Surgical technique brochures are available upon request. Before the initial use of the ORIO System, the surgeon should review all available information and consult with other surgeons having experience with these type of devices. The surgeon should be thoroughly familiar with the assembly of the components.

The implantation of two devices of the same size at each targeted level is recommended in case of posterior lumbar interbody fusion.

- **IMPLANTS FATIGUE.** Based on the fatigue testing results, the physician/surgeon should consider the levels of implantation, patient weight, patient activity level, other patient conditions, etc. which may impact the performance of the system.
- **PREVIOUS SPINAL SURGERY.** Patients with previous spinal surgery at the level(s) to be treated may have different clinical outcomes compared to those without a previous surgery.

7. PREOPERATIVE PLANNING AND POSTOPERATIVE CARE

Preoperative planning provides essential information regarding the appropriate implant and likely combinations of components. Use instrument trial components for fit verification (where applicable) and extra implants for backup. Accepted surgical practices should be followed for postoperative care. Excessive physical activity and trauma affecting the implanted devices have been implicated in premature failure by fracture, migration and/or wear of the implants. The patient should be cautioned to govern his/her activities accordingly as the risk of implant failure increases with weight and activity levels of the patient.

8. ADVERSE EFFECTS

- Cracking or fracture of the implants or loss of fixation in bone; attributable to nonunion, osteoporosis, markedly unstable comminuted fractures.
- Loss of anatomic position with malunion.
- Implant dislodgement or subsidence.
- Infections, both deep and superficial.
- Vascular or visceral injuries.
- Allergies and other reactions to device materials.
- Cracking or fracture of the vertebrae.
- Post-operative change in spinal curvature, loss of correction, and/or height reduction.
- Death.
- Patients with previous spinal surgery at the levels to be treated may have different clinical outcomes compared to those without a previous surgery.
- The risk of device expulsion and migration is higher without the use of supplemental fixation.

9. PACKAGING

Packages for each of the components should be intact upon receipt. If a loaner or consignment system is used, all sets should be carefully checked for completeness and all components including instruments should be carefully checked to ensure that there is no damage prior to use. Damaged packages or products should not be used, and should be returned to SpineCraft.

10. CLEANING AND DECONTAMINATION

Unless just removed from an unopened SpineCraft package, all instruments and implants must be disassembled (if applicable) and cleaned using neutral cleaners before sterilization and introduction into a sterile surgical field or (if applicable) return of the product to SpineCraft. Cleaning and disinfecting of instruments can be performed with aldehyde-free solvents at higher temperatures. Cleaning and decontamination must include the use of neutral cleaners followed by a deionized water rinse.

NOTE: Certain cleaning solutions such as those containing formalin, glutaraldehyde, bleach and/or other alkaline cleaners may damage some devices, particularly instruments; these solutions should not be used. Also, many instruments require disassembly before cleaning. All products should be treated with care. Improper use or handling may lead to damage and/or possible improper functioning of the device. Unless marked sterile and clearly labeled as such in an unopened sterile package provided by the company.

11. STERILIZATION

The PEEK-Optima® implants and instruments are provided nonsterile and should be stored in their original packaging until cleaned and sterilized according to the recommended guidelines listed below. Implants are single-use devices, thus do not clean or resterilize an implant that has been in contact with or contaminated by blood or other infectious substances. The manufacturer and distributor assume no responsibility for the cleaning and resterilization of implants, components, or reusable instruments performed by the individual or hospital. SpineCraft instruments and instrument cases are generally composed of titanium, stainless steel, aluminum, and/or polymeric materials. The cases may be multi-layered with various inserts to hold surgical instrumentation in place during handling and storage. The inserts may consist of trays, holders, and silicone mats. The instrument cases will allow sterilization of the contents to occur in a steam autoclave utilizing the cleaning, sterilization, and drying cycle that has been validated and listed below. Instrument cases do not provide a sterile barrier and must be used in conjunction with sterilization wrap to maintain sterility.

All implants and instruments used in surgery must be sterilized by the hospital prior to use. Remove all packaging materials prior to sterilization. Only sterile products should be placed in the operative field. Instructions for cleaning and sterilization of ORIO instruments can be found in SpineCraft publication # RG-0010-5 and can be obtained by contacting the company. Unless specified elsewhere, these products are recommended to be steam sterilized by the hospital using the set of process parameters below:

METHOD	CYCLE	TEMPERATURE	EXPOSURE TIME	DRY TIME
Steam	Pre-Vacuum	273° F (134° C)	18 min with Four Pulses	30 Minutes

Wrap: The wrap should be FDA cleared for the proposed cycle specifications.

Monitor every load with a PCD containing a BI and a Class 5 integrating indicator.

This Pre-Vacuum Steam sterilization is not considered by the Food and Drug Administration to be a standard sterilization cycle. It is the end user's responsibility to use only sterilizers and accessories (such as sterilization wraps, and sterilization cassettes) that have been cleared by the Food and Drug Administration for the selected sterilization cycle specifications (time and temperature).

NOTE: Because of the many variables involved in sterilization, each medical facility should calibrate and verify the sterilization process (e.g. temperatures, times) used for their equipment.

NOTE: SpineCraft does not recommend Flash Sterilization within instrument cases or Chemical Sterilization.

LIMITED WARRANTY AND DISCLAIMER: ORIO CAGE PRODUCTS ARE SOLD WITH A LIMITED WARRANTY TO THE ORIGINAL PURCHASER AGAINST DEFECTS IN WORKMANSHIP AND MATERIALS. ANY OTHER EXPRESS OR IMPLIED WARRANTIES, INCLUDING WARRANTIES OF MERCHANTABILITY OR FITNESS, ARE HEREBY DISCLAIMED.

IF MORE THAN TWO YEARS HAVE ELAPSED BETWEEN THE DATE OF ISSUE/REVISION AND THE DATE OF CONSULTATION, CONTACT SPINECRAFT FOR CURRENT INFORMATION at: +1 630-920-7300.

SURGICAL TECHNIQUE MANUAL COULD BE OBTAINED BY CONTACTING SPINECRAFT CUSTOMER SERVICE at +1 630-920-7300. ALSO, IT COULD BE DOWNLOADED DIRECTLY FROM THE COMPANY WEBSITE USING THE SURGEON LOG-IN.

SpineCraft
777 Oakmont Lane - Westmont, IL 60559, USA
TEL + 1 630 920 7300 - FAX + 1 630 920 7310
TF + 1 877 731 SPINE (+ 1 877 731 7746)

